

# ***User manual***

## ***Multifunctional skin care beauty machine***

*Model: SV-REGE*



## 一、***Product Introduction***

The oxygen-powered skin-beautifying instrument adopts the world's advanced PSA (pressure swing adsorption principle). When the power is turned on at room temperature, it can continuously separate high-concentration, dust-free and sterile oxygen from the air in an instant, and realize oxygen absorption at the same time , The three functions of oxygen supplement and atomized oxygen spraying are integrated. Completely and thoroughly solve the problem of lack of oxygen in the body and skin. Oxygen supplement usage is mainly through oxygen supplement tools, applying appropriate and limited pressure, using oxygen to inject the active ingredient products into the bottom of the skin, and then pushing through the pulse to deliver oxygen to deeper skin layers, effectively accelerating cells update restores a healthy, smooth, and radiant skin to a youthful state.

The multi-energy oxygen supplement instrument also combines the regeneration of BIO living cells, It has a good effect on deep cleansing of various skins, repairing damaged skin, renewing new cells, removing pigment spots, and improving red blood streaks. Make your skin natural, firm, delicate and shiny without any modification, truly turn back the time and restore youthful beauty.

## ⇒ *Principle*

1. **LED light mask:** Bioactive cold light beauty therapy (red, green, blue, yellow) known as the "beauty light" photodynamic is a biologically active light, It does not produce high heat, uses low-energy pure light, and the unique light-tuning skin care has the characteristics of natural gentleness and no side effects. It is the new darling of the beauty industry.



2. **Hydra handle:** Using the hydrodynamic circulation system, the ionized gene water directly enters the deepest layer of the pores, transports the nutrients in the water to the basal layer of the skin, and at the same time cleans the attachments on the inner wall of the pores comprehensively. Thoroughly clean the skin without making the pores enlarged, and at the same time it can replenish a lot of moisture to the skin, and it can supplement the moisture and basic nutrients needed by the cells. It is humanized with different densities of water sculpture mesh surfaces, suitable for any skin type, it completely replaces the micro-sculpted dermabrasion, crystal sand dermabrasion, oxygen supplement and hydration.



3. **water oxygen handle:** Using strong pressure oxygen, sterile pure water (normal pure water) and oxygen are sprayed out through a probe with a diameter of 5 microns at the same time, and water is quickly injected into the bottom of the skin.



4. **Ultrasound hand:** adopts sonic wave and medium wave to quickly introduce nutrients. Ultrasonic radio frequency can effectively penetrate the cell membrane to promote the division of collagen matrix and stains, so that high-efficiency nutrients can be quickly absorbed. Improve large pores and decompose skin oils,

**How to use:** *When using the sonic energy head, you should apply the product and operate it in a circular motion*



## 5. Ion eye pen:

Using carrier wave and radio frequency to exchange currents to make cell molecules generate heat, quickly stimulate the regeneration of collagen, achieve facial lifting and firming and eye wrinkles, eye bags, dark circles removal, the effect is better when used with eye cream and essence products , To promote acne repair, BIO micro-current massage can open electrophoresis, open pore channels, and promote product absorption

***Projects set up:*** *micro-current introduction, lymphatic drainage, lifting and tightening, removing dark circles*



6. **Cold hammer:** using the latest international K4 refrigeration system, the skin surface temperature can quickly reach minus 2 degrees Celsius, and the electroporation can be quickly closed, so that the nutrients entering the cell membrane are not lost, and the effect of removing redness and anti-sensitivity is achieved, and the damage is forced. The split stain, inhibits the life of pigment cells and achieves the effect of tenderness and whitening. Calmly repairs damaged skin, eliminates redness and swelling, shrinks pores, and relieves tired skin




7. **Nano spray gun:** oxygen supplement to add moisture to the skin. Enhance absorption and accelerate diffusion when imported into skin care products. Improve microcirculation, inhibit facial varices, improve dry skin and skin aging lines. Can improve cell viability, improve and promote Blood flow, accelerate update, and improve sensitive skin

***Applicable people:*** Especially suitable for urban smokers, mature women with dull, sensitive skin, wrinkles, spots and acne.



**8. Oxygen supplement handle:** oxygen and nutrient solution are sprayed out through the oxygen supplement gun, which quickly injects nutrients and oxygen into the skin to make the skin moisturized and vigorous.

**Use:** *Mainly essence (Y50 essence, anti-allergic repair essence, L-VC essence, eye essence, facial essence, neck essence), inject into the skin dermis Choose the right essence for different skins. Dry skin can achieve the effect of moisturizing and whitening, and oily skin can achieve the effect of shrinking pores and controlling oil and refreshing.*

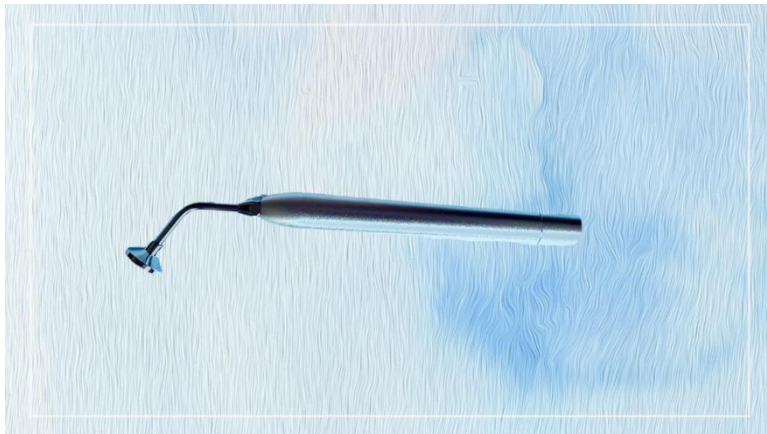

**9. V face ion wheel (warm):** When it acts on the skin, it can heat the collagen fibers in the dermis, shrink the collagen fibers, tighten the loose skin wrinkles, and stimulate the proliferation of collagen in the dermis to promote the dermis. The thickness and density of the skin are increased, so as to fill up wrinkles, restore skin elasticity and luster to achieve the purpose of firming and removing wrinkles.

Broadband electrophoresis technology directly penetrates the epidermis to 1.2-3.6mm below the skin, and accelerates the movement of cells with 2.45 billion vibrations per second. This bionic electrical effect will increase the gravitational force between cells, tighten muscles and cells, accelerate the movement of fat cells, liquefy and decompose the fat in the cells, drain excess water and toxins out of the body through lymphatic vessels, and restore muscles and cells. Tension. So as to improve the V face, shape the face, open the water protein channel at the same time, and improve the water transportation efficiency.




10. **Aerobic mask:** With non-woven fabric mask as the main combination, the skin can fully breathe and absorb oxygen, so that the effective ingredients can be fully absorbed.

**How to use:** Choose different essential oils and oxygen inhalation (use a mask, straw to do it) and apply SPA oxygen inhalation care.



11. **Skin detection:** examining the complexion of wrinkles, spots pores, texture, porphyrins, UV spots, pigmentation, acne moisture and skin age. There are 12 dermal and epidermal skin problems including spots, pores, moisture, texture, and wrinkles.

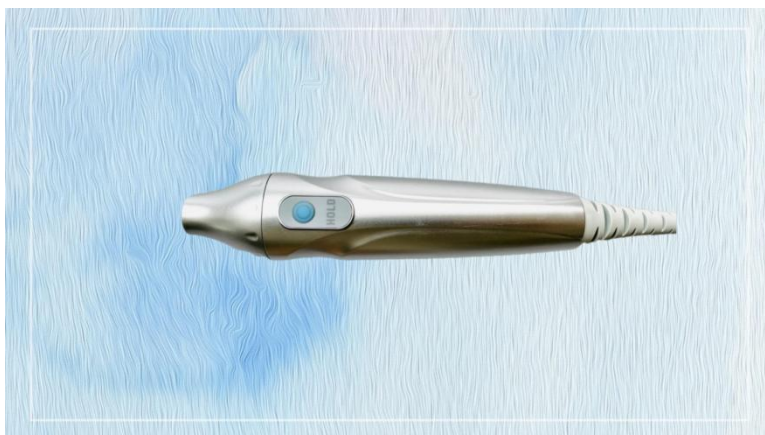

## ⇒ **FUNCTION**


1. Deep cleansing, acne and oil control, remove excess keratin and rubbish
3. Lifting and firming, photo rejuvenation, smoothing wrinkles
4. Whitening and hydrating, nutrition introduction,
5. Moisturizing and activating skin, full effect repair
6. Brighten skin tone and delay aging
8. Beauty and skin rejuvenation, calm and repair
9. Skin detection,


## ⇒ **Installation introduction:**

Install all the hangers and handles on the top of the back of the instrument.


**1. Water cup:** pure water, or add some original liquid or essential oil. The upper lid can be unscrewed to add water. Connect with the upper pipe.

**2. Waste bottle:** "After cleaning for the customer, some dirty things can be seen in the cup. Note: When the water level is 3/4 full, it must be cleared in time.

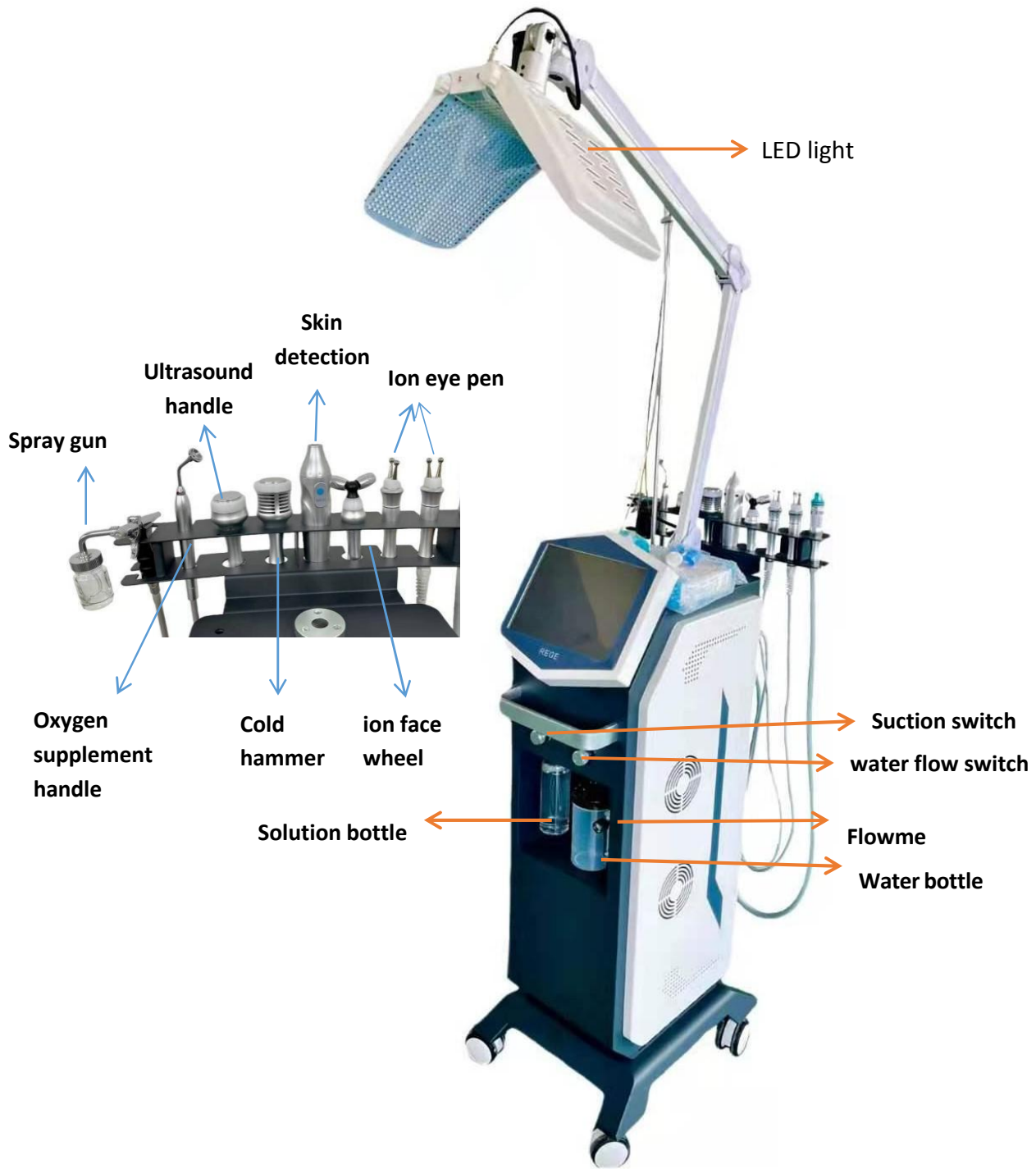

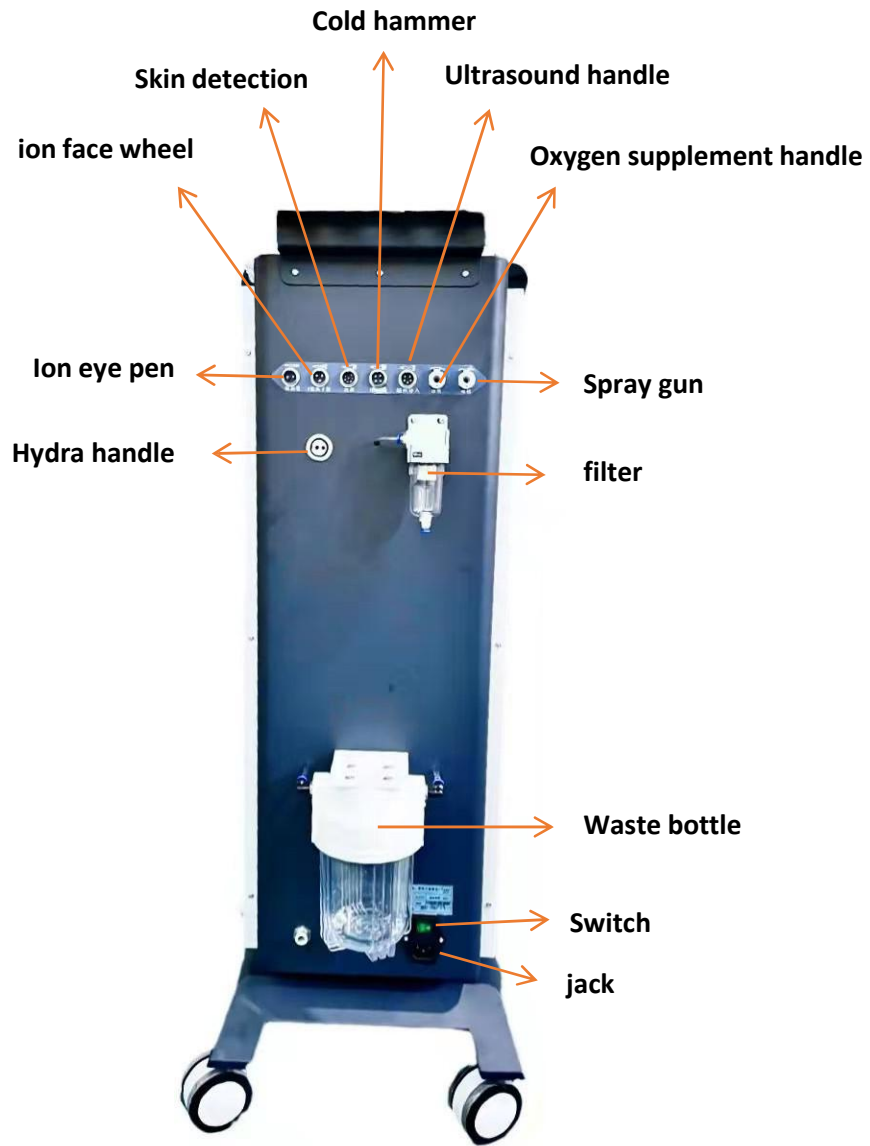

## Operation interface

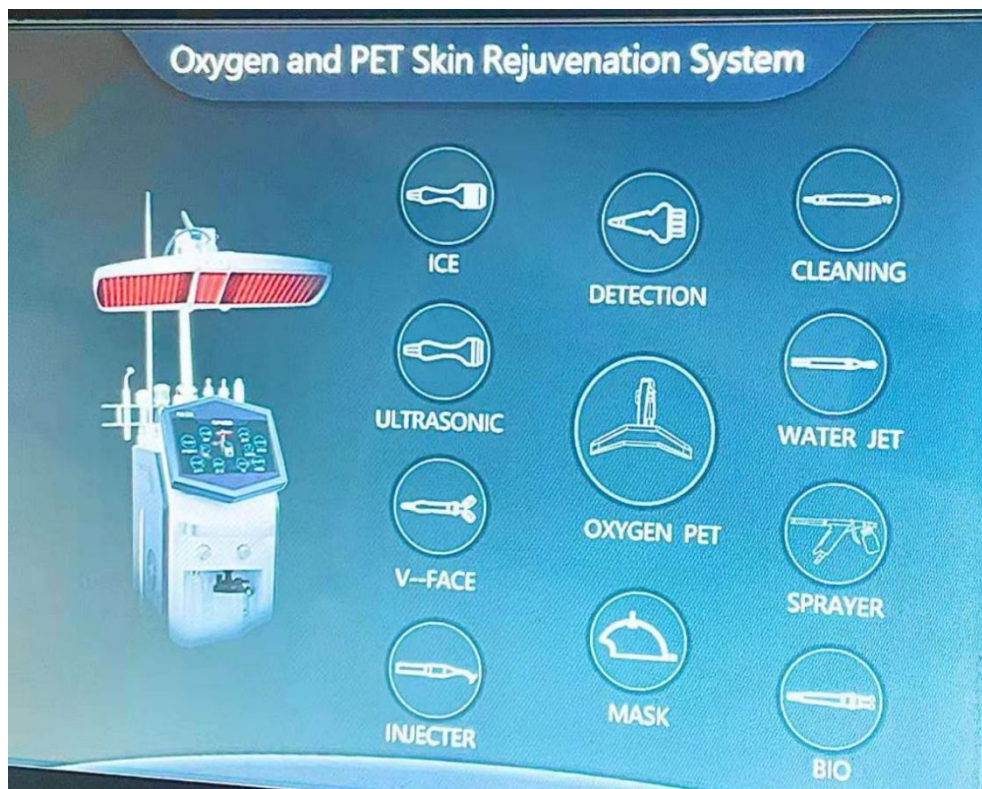

## Built-in 5 kinds of light sources to solve

### various skin problems




#### Mixed light:

It is a dual-frequency light of red light and blue light. It combines the effects of two kinds of light, especially in treating and removing acne marks and has a particularly good effect and repairing effect.

#### Blu-ray:

The blue light with a wavelength of 415nm has the effect of quickly inhibiting inflammation. During the formation process, the propionic acid bacteria are the main cause, and the blue light can destroy this kind of bacteria without any damage to the skin tissue. In a short period of time, the inflammation period is significantly reduced to full recovery

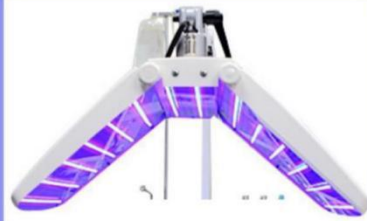


#### Green light:

It has the function of neutralization, balance and safety, relieves mental stress, and effectively dredges lymph and drainage

#### Yellow light:

It replenishes skin cells, promotes glandular function, assists in curing skin diseases, and enhances immune function.

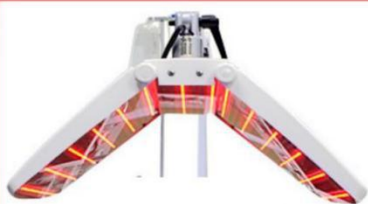

#### Red light:

Regenerate skin cells, improve rough pores, relieve acne marks and allergic dermatitis

## ***IV Operating procedures***

### **1. Oxygen supplement facial care**

Attack effect: Inject more than 98% oxygen and active ingredients into the basal layer of the epidermis, improve skin update, renew damaged cells, increase skin elasticity, improve dull and dull skin with pigmentation, slow down skin aging, and provide sufficient oxygen , Beautiful skin.

#### **Steps:**

- 1) Use the skin detection handle to detect the skin condition,
- 2) Clean facial skin with cleanser.
- 3) Thoroughly clean the skin, this step should be used in conjunction with the use of shovel to remove aging keratin and deep dirt (10 minutes)
- 4) Use a micro bubble pen to clean the facial water circulation, holding the pen and moving it along the skin. (15 minutes) Wipe clean the face with warm water.
- 5) Spray oxygen with essential oils. Take 2-3 drops each time and dilute with water. During the oxygen spray process, cooperate with the technique to improve. (15 minutes)
- 6) Oxygen supplement, apply the essence on the face, and use a pressure nozzle to perform neck massage, lifting, wrinkle removal and other techniques along the skin texture of the neck to inject oxygen. (15 minutes)
- 7) Use BIO skin lifter with essential oils to lift the neck, reduce neck wrinkles, and reduce muscle sagging and sagging. (15 minutes)

8) Apply a facial mask, put on an oxygen mask, and do oxygen-scenting care. (20 minutes)

9) Use a cold hammer to soothe and calm the skin

## **2. Hydra facial handle**

choose different crystal heads according to the customer's skin type or area. First, sterilize the two cups with alcohol, then fill the product container bottle at the back of the instrument with purified water, and tighten the lid of the waste bottle. Press the instrument interface switch, and then turn the air regulating valve to control the size of the water output and the size of the suction. (Operate with facial detoxification. If the water in the waste bottle is 2/3 full, the dirty water in the waste bottle must be drained before adding pure water to the water cup again to prevent the waste water bottle from being overfilled. Spill outside. Note: before the probe is soaked in alcohol, a guest must wash it in time after finishing it; those with more serious skin problems need to add a few drops of alcohol to the purified water to prevent skin inflammation; half of the face is operated for 5 minutes)

### ***Operation precautions:***

1. If the water circulation of the micro bubble head is interrupted, please check the following points:
  - a. Whether the air pressure is well adjusted;
  - b. Whether the pipe is intact and whether it is blocked.
  - c. Whether the sealing ring of the probe is clamped;
  - d. Whether the water supply in the water inlet cup is sufficient;

#### **4. Ultrasonic handle:**

Use the energy increase button to adjust the energy to 10 grids, and warm up for 20 seconds. The energy can be accepted as a suitable gear with the person being cared for. The upward lifting method shall prevail, and the line shall be the same as above. (The half-face usage time is 5 minutes, and the gear will be reset to zero after the operation)

#### **5. Cold hammer:**

first start the work area, then use the energy increase button to adjust the energy to 10 grids, pre-cool for 5 seconds, and then first test the temperature on the operator's hand, add or subtract according to the guest's adaptability, the operation method is the same as above Return the energy to zero after the operation (operation time is 5 minutes for half of the face)

#### **6. Ion eye pen:**

First select the BIO working area, and the energy will be strengthened from the first gear. When treating the eyes, it can be operated together with the facial decree lines, and the lifting technique is used. When the eyes are treated, because of the contact surface The smaller the energy, the stronger the energy. Remember: When acting on the upper eyelid, it must not be used on the eyeball part, and should be operated along the brow bone. (Single course of usage, one eye operation time is 5 minutes, then the energy is returned to zero)

## 五、 ***Precautions***

1. After the power is off, apply a small amount of detergent to the outside of the case with a clean soft wet towel, wipe the outside of the case, and then dry it with a dry towel. Wipe 1-2 times a month.
2. The upper cover of the humidification bottle must be tightened, otherwise it will leak and reduce the amount of oxygen. Brush once a day to prevent bacteria from growing in the water.
3. Keep the oxygen supplement device away from the fire source.
4. The parts in contact with oxygen such as oxygen atomizers must be free of grease to avoid accidents.
5. Do not release oxygen in a closed environment to avoid accidents.
6. It is strictly forbidden to smoke or light a fire when the machine is started. Do not spray oxygen on flammable liquids, as this may cause accidents.
7. The use of this instrument shall not exceed two hours each time.
8. When the water and oxygen function of the instrument is not in use, take out the pure water bottle and remove the water and oxygen tube from the main unit to prevent the backflow of pure water from damaging the main unit.

## 六、 ***Product Service and Repair***

This machine is guaranteed for one year from the date of purchase (by receipt), and there is no need for quality problems within one year.

Repairs are required, and repairs will be charged if there is a problem after one year. The spare parts are repaired free of charge for three months, and the spare parts repair fee will be charged after three months. If the warranty period has expired, it can also be responsible for maintenance and charge the cost of spare parts and consumables.

Free warranty service is not provided for the following failures caused by personal reasons:

1. Failure caused by unauthorized disassembly or modification of this product;
2. Failure caused by accidental use and fall;
3. Failure caused by lack of reasonable maintenance;
4. Failure caused by not following the correct instructions in the instructions for use.

### ***七 Packing list:***

	Parts name	unit	Quantity
1.	Host machine	Piece	1
2.	Cable line	piece	1
3.	Aerobic mask	piece	1
4.	Oxygen supplement handle	piece	1
5.	Spray gun	piece	1
6.	Hydra handle	piece	1
7.	Ion eye pen	piece	2
8.	Cold hammer	piece	1
9.	Ultrasound handle	piece	1
11.	Water oxygen pen	piece	1
12.	Ion fave pen	piece	1
13.	Hanger	piece	1
14.	Waste bottle	piece	1
15.	Solution bottle	piece	1
16.	LED Mask	piece	1
17.	Nasal mask	piece	1
18.	Sniffing tube	piece	1
19.	User manual	piece	1

***Technical parameter table:***

Rated power: 250W

Working voltage: 220V/50HZ

Oxygen flow: 1-3L/min

Oxygen concentration: >93%

Machine noise: ≤45db